Tizian Block 5.0

SHÜTZ DENTAL Micerium Group	<small>REF 137 Rev 00 Print Date: 08.02.2019</small>
Tizian Block 5.0 Multilayer Zirconia	
US-EN Tizian Block 5.0 Multilayer Zirconia Cubic Dental Zirconia (5Y-FSZ)	
DE-ES Tizian Block 5.0 Multilayer Zirconia Dióxido de zirconio dental (5Y-FSZ)	
GB Tizian Block 5.0 Multilayer Zirconia Cubic Dental Zirconia (5Y-FSZ)	
DE Tizian Block 5.0 Multilayer Zirconia Kubisches Dental-Zirkoniumdióxido (5Y-FSZ)	
FR Tizian Block 5.0 Multilayer Zirconia Dióxido de zirconium dentaire (5Y-FSZ)	
IT Tizian Block 5.0 Multilayer Zirconia Biossido di zirconio (5Y-FSZ)	
ES Tizian Block 5.0 Multilayer Zirconia Dióxido de zirconio dental (5Y-FSZ)	
RO Tizian Block 5.0 Multilayer Zirconia Dioxid de zirconiu dental (5Y-FSZ)	
RU Tizian Block 5.0 Multilayer Zirconia Стomatологический диоксид циркония (5Y-FSZ)	
US-EN For use in the United States	

INSTRUCTIONS FOR USE

Tizian Block 5.0 Multilayer Zirconia | Cubic Dental Zirconia (5Y-FSZ)

This device is intended to be sold only for processing by trained persons, milling laboratories or centers authorized to process dental prostheses.

INTRODUCTION
Please read these instructions completely and thoroughly before using the product and observe them at all times. Improper use of the product and failure to observe the information provided may affect quality and reduce the lifetime of the dental prosthesis. The product must be used in accordance with these instructions and only for the specified purpose for which it was designed. **Schütz Dental GmbH / pridentia GmbH** shall not be liable for any consequential damages or injury arising from any use or misuse of this product. In using **Schütz Dental GmbH's / pridentia GmbH's** product you assume the responsibilities of ownership and operation, and you agree to hold **Schütz Dental GmbH / pridentia GmbH** harmless for any incident or action involving the use of **Schütz Dental GmbH's / pridentia GmbH's** product. Please keep these instructions in a safe location for further reference over the entire lifetime of the product. Pass on the information provided to any future owner, processor or user of this product or of further processed products made from it.

Please be aware of the different types of risk involved in the use of the product:

- WARNING** indicates a hazardous situation that, if not avoided, may result in minor or moderate injury or in property damage.
- CAUTION** indicates a hazardous situation that, if not avoided, could result in serious or moderate injury or in property damage.

PRODUCT DETAILS
Block, dimensions: 40 mm x 19 mm x 15.5 mm
multilayer: A light (shade range: A1-A3), A dark (shade range: A3.5-A4), B light (shade range: B1-B2)

PRESCRIPTION ONLY
Type II Class 4 medical device in accordance with DIN EN ISO 6872:2019

INDICATIONS FOR USE
Tizian Block 5.0 Multilayer Zirconia are pre-sintered zirconium oxide blocks for use in CNC (Computer Numerical Controls) milling machines to fabricate partial and single anterior and lateral crowns, bridges up to 3 units, inlays and veneers for dental prostheses for restoration purposes only.

GENERAL INFORMATION
Check the delivery immediately on receipt to ensure that:

- all components are present
- the packaging and the product are intact

WARNING: Do not use if there are any cracked, fractured or broken areas or discolorations visible on the product. If any defect is identified, the blank must not be used for fabrication of dental prostheses. Processing a cracked, fractured, broken or discolored product may result in a defective restoration with a risk of injury to patients.

If you detect any defect in the product, please contact your dealer or the manufacturer. Should there arise particular problems which are not covered sufficiently by these instructions, the matter should be referred to the manufacturer. The manufacturer accepts no liability for defective results since it has no influence on the sintering process.

STORAGE CONDITIONS
Tizian Block 5.0 Multilayer Zirconia blocks should be stored in their original packaging. Ensure that:

- they are stored in a dry place
- temperatures are between 5 °C (41 °F) and 50 °C (122 °F)
- they are not subject to excessive vibration

CAUTION: Do not store in a damp area. Dampness may affect the product. Do not store near potential contamination sources as they may affect the product.

MATERIAL PROPERTIES
Tizian Block 5.0 Multilayer Zirconia blocks meet the following specifications at completion of the high-density sintering process.

Material characteristics:
Flexural strength: > 650 MPa
CTE: 10 ·10⁻⁶ K⁻¹
Chemical composition:
Zirconium dioxide (ZrO₂H₂O): 89.89 - 90.7%
Yttrium oxide (Y₂O₃): 8.55 - 10.11%
Aluminum oxide (Al₂O₃): < 0.2%
Other oxide: < 0.7%

The proportions of the single components in the total quantity of the components in a block may vary within the ranges described above; however, the total quantity of components in each single block will not exceed 100%.

MACHINING
As **Tizian Block 5.0 Multilayer Zirconia** is made of a sensitive high performance material, always handle it with care. The contact with liquids should be avoided except for the CNC machining process. This product must only be managed by trained technicians. The safety warnings in these Instructions for use must be complied with. Users bear sole responsibility for the use of the product. The manufacturer accepts no liability for defective results since it has no influence on the machining process.

CONSTRUCTION				
Systems must be constructed of the following specifications:	Crowns	Maryland-bridges	3-unit bridges	
Minimum framework thickness	anterior 0.4 mm posterior 0.6 mm	0.4 mm 0.6 mm	0.6 mm 0.6 mm	
Connectors	anterior - posterior 9 mm²	6 mm² 9 mm²	6 mm² 9 mm²	
Framework design	Anatomically shaped (supporting veneer ceramics), fully anatomical			

GRINDING AND MILLING

CNC (Computer Numerical Control) processing machines, machine tools and processing parameters must be used for grinding and milling the blocks. You are strongly advised not to use coolant during the grinding and milling process as it may lead to color changes and/or loss of transparency. Check the grinding or milled product for discoloration, fissures and cracks after processing. Use only grinding and milling systems that are recommended by their manufacturers for the processing of zirconium oxide. Grinding and Milling systems need to be well calibrated for best results. All systems are not alike and can produce adverse results if the minimum thickness is not followed. Because the material shrinks during sintering, it is vital to take into account the appropriate shrinkage factor during grinding and milling to ensure the precise fit of the restoration. Each block is labeled with the specific shrinkage factor that has to be applied.

WARNING: Dust from the milling or grinding process or from pre-sintering manual adjustments may cause irritation of eyes, mucous membranes, skin or damage to lungs. Processing may therefore be performed only with a properly functioning aspirator, safety glasses and an approved dust mask. The manufacturer accepts no liability for defective results since it has no influence on the milling process.

SINTERING
All restorations made of **Tizian Block 5.0 Multilayer Zirconia** must go through a final sintering cycle prior to finishing. The sinter firing must only be conducted in high-temperature ovens authorized for this purpose. You may use the firing process specified by **Schütz Dental GmbH**. Please also note the oven manufacturer's specifications. All standard programmable dental laboratory ovens may be used.

Sintering ovens vary in their performance, so it is highly recommended the users calibrated periodically to ensure the recommended cycle is carried out correctly. Follow the manufacturer's recommended calibration instructions.

WARNING: Sintering oven must be located in a fireproof well-ventilated area. Do not open the oven or remove the sintered restoration before the furnace has cooled down sufficiently for safe handling to avoid the risk of burns.

Recommended firing program:
From room temperature to 1.450 °C at 10 °C/min, hold time of 2 hours at firing temperature, allow to cool to room temperature at 10 °C/min. Natural cooling begins from approx. 600 °C depending on the oven used.
Rapid sintering (optional):
Rapid sintering may be conducted subject to the following conditions:
Crowns and bridges with up to 3 units: From room temperature to 1.500 °C at 10 °C/min, hold time of 30 minutes, allow to cool to room temperature at 40 °C/min.

WARNING: Always comply with the above sinter cycles, otherwise the material may become weakened and may break in the patient's oral cavity.

You are strongly advised not to use coloring liquids as this can have a negative effect on translucency or shade. If restorations have been fabricated in the sintering oven using liquid stains, a cleaning firing with zirconium oxide powder is recommended before **Tizian Block 5.0 Multilayer Zirconia** is sintered in the same oven.

CAUTION: Do not use undesirable color change when sintering pre-shaded zirconium oxide. It is strongly advised to insert a spacer (made of zirconia) of at least 1 mm height between the lid and the tray to ensure that the air can circulate.

Check the sintered product for discoloration, fissures and cracks after processing.

WARNING: Do not use a cracked, fractured, broken or discolored product may result in a defective restoration with a risk of injury to patients.

If you detect any defect in the product, please contact your dealer or the manufacturer. Should there arise particular problems which are not covered sufficiently by these instructions, the matters should be referred to the manufacturer. The manufacturer accepts no liability for defective results since it has no influence on the sintering process.

ADJUSTMENTS
If adjustments to the densely-sintered restorations are required, these must only be conducted using water-cooled diamond-tipped grinding tools in order to avoid damage of the materials by local overheating or excessive force on the surface of the restoration. Never use milling tools as they will damage the surface of the restoration.

Basic rules for handling sintered materials:

- work at low pressure.
- use diamond-tipped grinding tools in good condition only.
- avoid sharp edges of the finished product to prevent patient injuries.
- do not machine interdental connectors.

The **Tizian Block 5.0 Multilayer Zirconia** is a final restoration. It is not intended to be used for additional adjustments. The manufacturer accepts no liability for defective results since it has no influence alguna en el proceso de fessado.

CERAMIC VENEERS
All veneer ceramics recommended for zirconium oxide ceramics may be used.

FIXING

Tizian Block 5.0 Multilayer Zirconia restorations may be cemented conventionally with a zinc phosphate or glass ionomer cement or with self-adhesive fixing and full-adhesive material. Ensure that surfaces are retentive and that stump height is at least 3 mm.

WARNING: Do not use a final restoration if there are any cracked, fractured or broken areas or discolorations visible on the product. If any defect is identified, the framework must not be used on a patient. Otherwise you will risk injuries in the patient's oral cavity or the inhalation of the product or parts of it.

CONTRAINDICATIONS

WARNING: Do not use the restoration

- for the fabrication of implants
- for the fabrication of abutments

- on patients with parafunctional habits (e.g. bruxism)
- on patients with a known intolerance to components
- for bridge restorations with 4 units and more
- with inadequate preparation
- with insufficient space available in the patient's mouth
- on patients with inadequate oral hygiene

Otherwise you will risk severe patients' injuries.

TECHNICAL SUPPORT

Manufacturer: pridentia® GmbH, Meisenweg 37, 70771 Leinfelden-Echterdingen, Germany

Worldwide Distributor: Schütz Dental GmbH, Dieselstr. 5-6, 61191 Rosbach, Germany, Phone +49 (0) 6003 814 - 0, Fax +49 (0) 6003 814 - 906, www.schutz-dental.de, info@schutz-dental.de

Explanation of symbols						
	Manufacturer	Use-by date	Keep-dry	Catalogue number	Temperature limit	Incisal / Occlusal
	Consult the instructions for use	Batch Code	Prescription only	Conformité Européenne	Identification number of notified body	

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