PRODOTTI NUOVI/NEW PRODUCTS/NUEVOS PRODUCTOS

SHINY DIAMOND POLISHER

Ref	Descrizione	Description	Descripción	Q.tà/Q.ty/C.dad
CS014	SHINY14 Gommini diamantati bianchi prelucidatura composito	SHINY14 Diamond Rubber white for pre-polishing composite	SHINY14 Gomas diamantadas blancas para pre pulir el composite	6 pz/6 pcs/6 uds
CS015	SHINY15 Gommini diamantati grigi per lucidare composito	SHINY15 Diamond Rubber grey for polishing composite	SHINY15 Gomas diamantadas gris para pulir el composite	3 pz/3 pcs/3 uds

SHINY IMPREGNATED BRUSH (Si Ca)

Ref	Descrizione	Description	Descripción	Q.tà/Q.ty/C.dad
CS041	SHINY41 Spazzolini impregnati per posteriori a punta	SHINY41 Impregnated Brush for posterior "Point"	SHINY41 Cepillo impregnado para posteriores a punta	6 pz/6 pcs/6 uds
CS042	SHINY42 Spazzolini impregnati per posteriori a mini coppetta	SHINY42 Impregnated Brush for posterior "Mini Cup"	SHINY42 Cepillo impregnado para posteriores en forma de pequeña copa	6 pz/6 pcs/6 uds
CS043	SHINY43 Spazzolini impregnati per posteriori a coppetta media	SHINY43 Impregnated Brush for posterior "Cup"	SHINY43 Cepillo impregnado para posteriores en forma de media copa	6 pz/6 pcs/6 uds

SHINY DIAMOND STRIPS

Ref	Descrizione	Description	Descripción	Q.tà/Q.ty/C.dad
CS050-2R	Strisce diamantate grana fine rossa 12mm x 2mm	Diamond strips fine red 12mm x 2mm	Bandas diamantadas de lija fina roja 12mm x 2mm	5 pz/5 pcs/5 uds
CS050-2Y	Strisce diamantate grana extra-fine gialla 12mm x 2mm	Diamond strips fine yellow 12mm x 2mm	Bandas diamantadas de lija extra-fina amarilla 12mm x 2mm	5 pz/5 pcs/5 uds
CS050-4R	Strisce diamantate grana fine rossa 12mm x 4mm	Diamond strips fine red 12mm x 4mm	Bandas diamantadas lija fina roja 12mm x 4mm	5 pz/5 pcs/5 uds
CS050-4Y	Strisce diamantate grana extra-fine gialla 12mm x 4mm	Diamond strips fine yellow 12mm x 4mm	Bandas diamantadas de lija extra-fina amarilla 12mm x 4mm	5 pz/5 pcs/5 uds

SHINY14 Gommino diamantato coarse

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SHINY 14 Diamond coarse polisher

It is suggested to be used **instead of SHINY 13 to pre-polish HRI Universal Enamel** (Shiny 13 is still suggested for HFO Generic Enamel), before using Diamond paste as described in Shiny polishing instructions manual.

- **To extend the life** of SHINY 14, and for best results, use slight to moderate pressure only.
- Max. speed: **7,500 rpm**.
- **Cooling:** Use in conjunction with water spray to reduce heat production.

SHINY 15 Diamond fine polisher

- **To Polish Composite:** Using a slow speed handpiece between 8,000-15,000 RPM, apply **light pressure** with intermittent water spray in a wiping motion. In just 30-60 seconds, a high gloss finish will appear.
- **To shape and polish composite:** Using a slow speed handpiece at between 8,000-15,000 RPM, with water spray, apply **moderate to heavy pressure** in areas requiring re-shaping, continuously checking surfaces to achieve desired results. After contouring the composite, apply only light pressure with intermittent water spray until a high gloss finish is achieved.
- **Note:** Contouring composite will degrade the polishers more quickly than polishing alone.

SHINY 41-42-43 Impregnated Brushes (Si Ca)

These impregnated brushes available in several shapes are ideal to polish composite in posterior teeth.

- Max. speed: **5,000 rpm**.

SHINY 50 Diamond Strips

These diamond strips available in 2 thicknesses and 2 widths are ideal to finish composite in interproximal areas

CLEANING: Automatic: in ultrasonic cleaner min. 15 minutes. Manual: cleaning agent validated for burs, disposable tissue, running water.

STERILIZATION: Sold unsterilized. Reusable. Sterilize before each use in autoclave at 134°C/273°F min. 3 minutes.

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